


SideStream Plus




Read the instructions for use booklet before using SideStream Plus for the first time.


How to use your SideStream Plus nebulizer

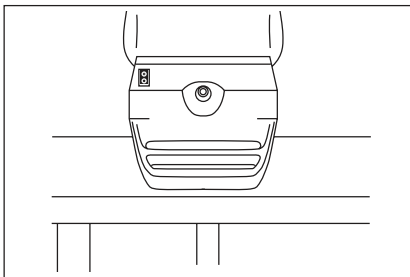


1. Place compressor on a solid surface.

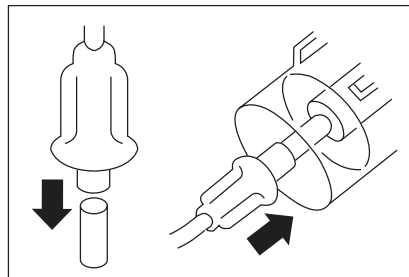


2. Connect tubing to the compressor and medicine cup.

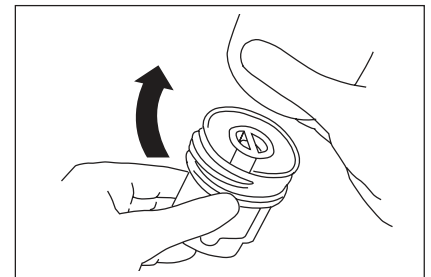


3. Unscrew lid.

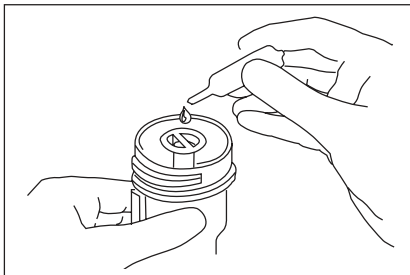


4. Fill medicine cup.

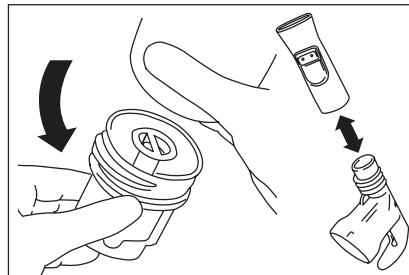


5. Replace lid. Connect mouthpiece.

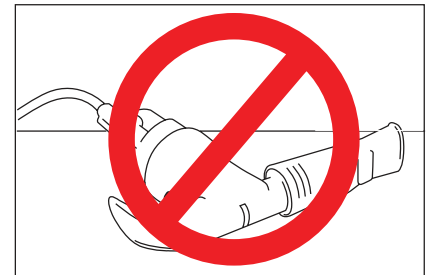


6. Do not tip SideStream Plus.

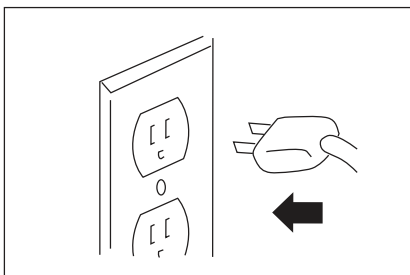

7. Plug compressor into outlet.

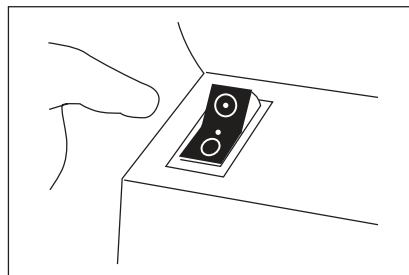


8. Turn switch on.

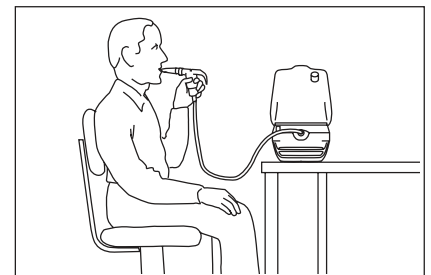


9. Sit upright, relax, and slowly breathe in and out.


If you need to use a SideStream Plus adult or pediatric mask ...

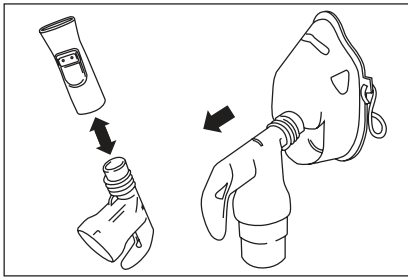

1. Remove mouthpiece and attach mask.

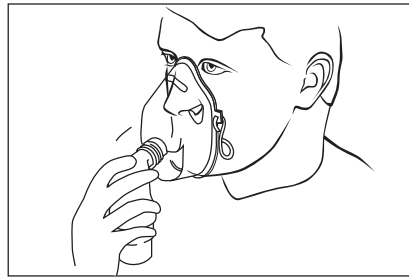


2. Place mask over nose and mouth.

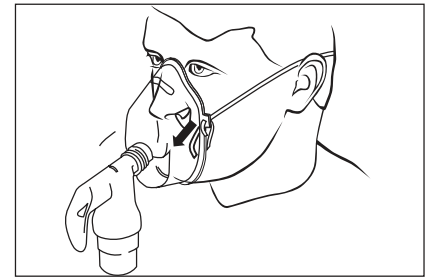


3. Place strap over head and adjust.

Cleaning instructions


After every use...

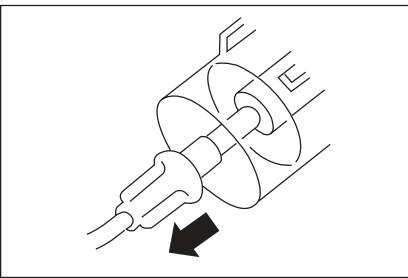

1. Disconnect handset from tubing.

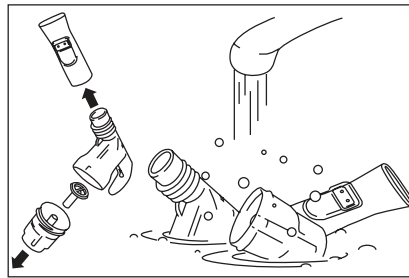

2. Disassemble and wash all parts in warm, soapy water.

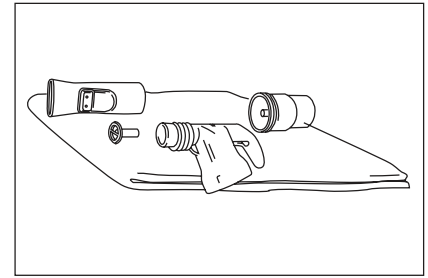


3. Rinse in clean water and air dry.


Once each week...




1. Place parts (except tubing) in water and 2-3 drops of dishwashing liquid.

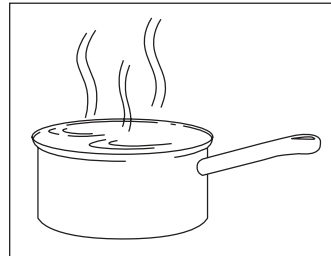

2. Boil 10 minutes.

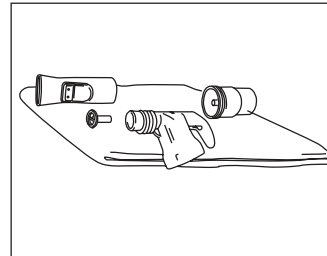

3. Rinse in clean water for 2 minutes and air dry.

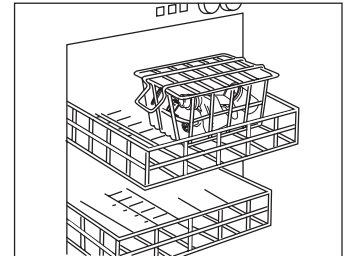

4. May be washed once a week on top rack of dishwasher, within basket.

Replace SideStream Plus after six months of use

If you have any questions regarding this information, please contact Philips Respironics Customer Service at (800) 962-1266.

Respironics and SideStream are trademarks of Respironics, Inc. and its affiliates.

© 2009 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication. CAUTION: US federal law restricts these devices to sale by or on the order of a physician.

DD MCI 4102297 PN 1061123

Philips Healthcare is part of Royal Philips Electronics

Europe, Middle East, Africa:
+33 1 47 52 30 00

Asia Pacific: +852 3194 2280

www.philips.com/respironics

Philips Respironics
5 Wood Hollow Road
Parsippany, NJ 07054
+1 800 345 6443
+1 724 387 4000

